

<p>Medium/Low level protection</p>  <p>TrellChem® Splash 600</p>	<p>EN 14605 type 4 coverall eg. Splash 600 with sock or attached boots, attached gloves, full face mask (P3 filter or SCBA)</p>
--	---

Note

What level of protection is needed must be decided by the user based on a **risk analysis** (type of work, risk of contact, duration of work, etcetera).

Guidelines for decontamination of Virus and Bacteria

A Trelchem suit that has been in contact with Virus and/or Bacteria can be decontaminated by using **normal disinfectants**, commonly used in hospitals. Other decontamination solutions containing **formaldehyde, chlorine or iodine** is also effective.

A disinfectant/decontamination solution does not kill the micro-organisms instantly, it **takes some time**. Therefore the solution should always stay on the suit for a period of time before washing it off. A **higher concentration of the disinfectant/decontamination solution** is more efficient.

Examples of disinfectant/decontamination solutions

- Hydrogen peroxide, 3 % in 15 minutes or
- Calcium hypochlorite 5 – 10 % in 2 – 4 hours

After decontamination wash the suit with soap and water.

For more info, contact us

Ansell
Johan Kocksgatan 10
SE-231 81 Trelleborg
Sweden
Tel. +46 (0) 10 205 1800
Fax. +46 (0) 10 205 1840
info@ansell.eu
www.ansell.com