

Personal Protective Equipment by Ansell		Natural Rubber Latex			Synthetic	
		Sterile/Handspecific			Non-Sterile	
		Gammex® PF XP (NRL)	Gammex® PF XP™ tested after alcohol desinfection	DermaShield® (Neoprene)	NitraTouch® (Nitrile)	* NitraTex® & NitraTex® EP (Long Cuff) (Nitrile)
15 min	Carmustine	3	3	4	2	3
	Cisplatin	6	6	6	3	6
	Cyclophosphamide	6	6	6	4	6
	Cytarabine	6	6	6	6	6
	Docetaxel	6	6	6	6	6
	Doxorubicin	6	6	6	6	6
	Etoposide	6	6	6	6	6
	5-Fluorouracil	6	6	6	3	6
	Ifosfamide	5	5	6	4	4
	Irinotecan	6	6	4	4	5
	Methotrexate	6	6	6	4	6
	Mitomycin C	6	6	6		
	Oxaliplatin	6	6	6		
	Paclitaxel	6	6	6		
	Thiotepa	6	6	6	5	6
	Vinorelbine	6	6	4	6	4
Ganciclovir *	6	6	6			
30 min	Carmustine	2	2	3	2	3
	Cisplatin	3	3	6	3	6
	Cyclophosphamide	5	4	5	4	4
	Cytarabine	5	4	4	6	6
	Docetaxel	5	5	5	5	5
	Doxorubicin	6	5	6	4	6
	Etoposide	6	6	6	2	6
	5-Fluorouracil	6	6	6	3	6
	Ifosfamide	4	4	4	4	4
	Irinotecan	4	4	4	4	5
	Methotrexate	6	6	5	4	5
	Mitomycin C	6	6	4		
	Oxaliplatin	4	4	6		
	Paclitaxel	4	4	4		
	Thiotepa	5	5	5	5	6
	Vinorelbine	6	6	4	6	4
Ganciclovir *	5	5	5			
60 min	Carmustine	1	2	3	2	3
	Cisplatin	3	3	6	3	6
	Cyclophosphamide	5	5	5	4	3
	Cytarabine	5	5	4	6	5
	Docetaxel	5	5	5	4	5
	Doxorubicin	5	5	6	4	6
	Etoposide	4	4	4	2	6
	5-Fluorouracil	6	6	6	3	6
	Ifosfamide	4	4	4	4	4
	Irinotecan	4	4	4	3	5
	Methotrexate	6	6	5	4	5
	Mitomycin C	5	5	4		
	Oxaliplatin	4	4	4		
	Paclitaxel	4	4	4		
	Thiotepa	4	4	5	5	5
	Vinorelbine	6	6	4	5	4
Ganciclovir *	4	4	4			

Permeation temperature indicator: 25°C 37°C 43°C

*NitraTex® is a medical device for the delivery to the patient

Key to Protection level

Level 1 : Permeation detected up to 10 times below EN374-3 limit requirement

Level 2 : Permeation detected 10 to 100 times below EN374-3 limit requirement

Level 3 : Permeation detected 100 to 1.000 times below EN374-3 limit requirement

Level 4 : Permeation detected 1.000 to 10.000 times below EN374-3 limit requirement

Level 5 : Permeation detected 10.000 to 100.000 times below EN374-3 limit requirement

Level 6 : > Permeation detected at least 100.000 times below EN374-3 limit requirement or no permeation measured